

For further help and advice

Phone: 18001 0800 7311 888 (textphone)
0800 7311 888 (telephone)

Email: helpline@rnid-typetalk.org.uk

Fax: 0151 709 8119

www.typetalk.org

RNID Typetalk
Harrington Road
Brunswick Business Park
Liverpool
L3 4DF

In an emergency dial **18000**

Text users calling from abroad - +44 151 494 1260
Hearing users calling from abroad - +44 151 494 2022

The Customer Service Team are available Monday -
Friday 9am-8pm, Saturday and Sunday 9am-5pm.

This leaflet is available in large print and braille.

The Royal National Institute for Deaf People. Registered Office: 19-23 Featherstone Street, London EC1Y 8SL. A company limited by guarantee registered in England and Wales No. 454169, Registered Charity Numbers 207720 (England and Wales) and SC038926 (Scotland).



User Guide: minicom 7000 sending ID / auto ID

RNID Typetalk

Minicom 7000

Senidn ID / Auto ID

1. Turn on your M7000
2. HOLD down CTRL and PRESS Number 9/OPTIONS at the same time
3. You will see 'PRINT (UP/LOW) CASE'
4. Press RETURN until you see 'AUTO ID (ON)
5. Press the ANNOUNCER (spacebar) you will see AUTO ID (OFF)
6. Press ESC
7. Turn off the Minicom

Minicom 7000

Seek time

1. Turn on your M7000
2. HOLD Down CTRL and PRESS Number 9/OPTIONS at the same time
3. You will see 'PRINT (UP/LOW) CASE'
4. Press RETURN until you see 'CCITT SEEK TIME'
5. Press NUMBER 9
6. Press ESC
7. Turn off the Minicom



Certificate No. 547



INVESTOR IN PEOPLE